



Model K

There are traces of powder at the air pinch valve.

A possible cause of this problem is a ruptured air pinch valve tube. Inspect and replace tube if necessary.

With the powder switch on, the red indicator does not activate.

There are two possible causes of this problem.

1. Your powder switch is defective. Replace.
2. You aren't getting air pressure to the unit. Check your air supply.

The powder spray from the nozzle is not well defined. How do I adjust this?

You are operating with a worn nozzle. Replace the nozzle.

My vibrator is inoperative. What has caused this?

There are two possible causes of this problem.

1. You have an open rectifier. Inspect and replace if necessary.
2. You have an open rheostat. Inspect and replace if necessary.
3. The magnetic coil of the vibrator is burnt out. Replace the vibrator assembly.